


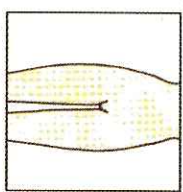


Incontinence Application Technique - Moliform


























Mobile



Chair fast

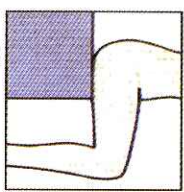

Bed fast

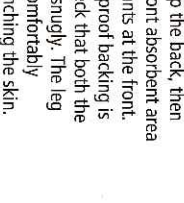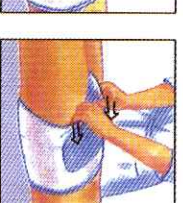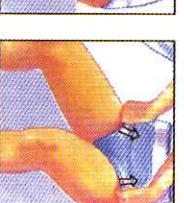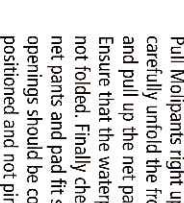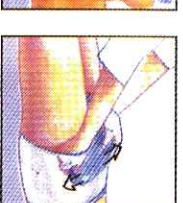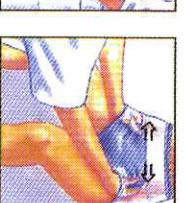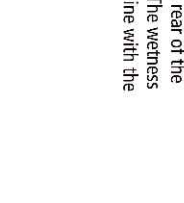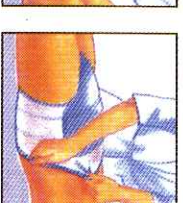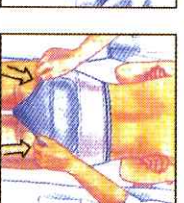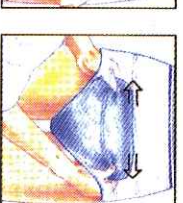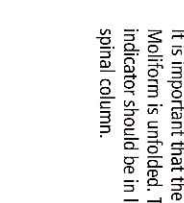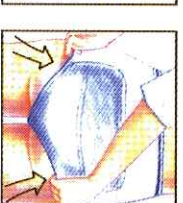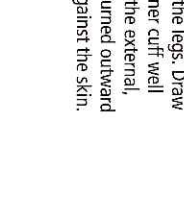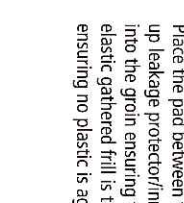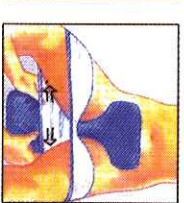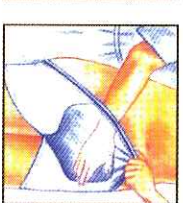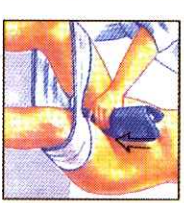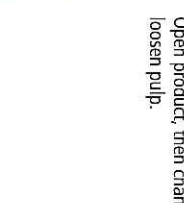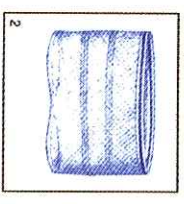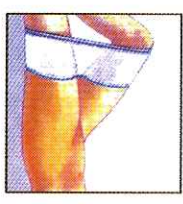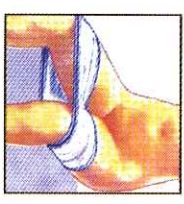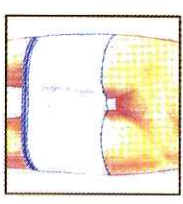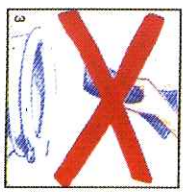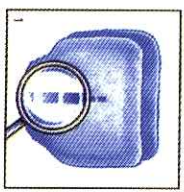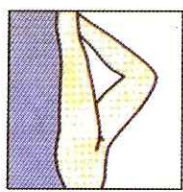

Open product, then channel pad and loosen pulp.

Place the pad between the legs. Draw up leakage protector/inner cuff well into the groin ensuring the external, elastic gathered frill is turned outward ensuring no plastic is against the skin.

It is important that the rear of the Moliform is unfolded. The wetness indicator should be in line with the spinal column.

Pull Molipants right up the back, then carefully unfold the front absorbent area and pull up the net pants at the front. Ensure that the waterproof backing is not folded. Finally check that both the net pants and pad fit snugly. The leg openings should be comfortably positioned and not pinching the skin.

1. The word Moliform in the crotch area is also the wetness indicator. When it disappears, the product needs changing. Always remove the used product from the back.
2. Elastic net Molipants are ideal for keeping Moliform Incontinence Pads securely in position.
3. Warning on disposal: Never throw the product into the W.C. after use.

THE HARTMANN ALLIANCE

NEW SOUTH WALES

Paul Hartmann Pty Ltd
Phone: 1800 805 839

VICTORIA & TASMANIA

Sutherland Medical Pty Ltd
Phone: 1300 664 027

QUEENSLAND

Medical & Surgical Requisites Pty Ltd
Phone: 1800 801 226

SOUTH AUSTRALIA & NT

McNeil's Surgical Pty Ltd
Phone: 08 8363 0888

WESTERN AUSTRALIA

Perth Surgical Supply Co. Pty Ltd
Phone: 08 9344 3111

© Registered trademark. Paul Hartmann AG ABN 35 000 099 589. Unit 27-28 Homeshush Business Village 11-21 Underwood Road Homeshush NSW 2140.